

HI-CON 128 NPD

NEUTRAL DISINFECTANT AND DETERGENT

Disinfectant, Fungicide, Mildewstat, Virucide*

DESCRIPTION

A highly concentrated, multi-purpose hospital grade germicidal detergent and deodorant. Effective at 1 ounce per gallon in hard waters up to 400ppm, plus 5% organic serum. The product is pseudomonacidal, staphylocidal, salmonellacidal, fungicidal, and virucidal. Effective against HIV-1 (AIDS Virus).

NOTE: Before using these compounds, food products and packaging materials must be removed from the room or carefully protected. After using these compounds, surfaces must be thoroughly rinsed with potable water before operations are resumed. The compounds must always be used at dilutions and according to applicable directions provided on the EPA registered label.

DIRECTIONS

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. For use on hard non-porous surfaces such as floors, walls, metal surfaces, porcelain, and plastic surfaces. Remove gross filth and heavy soil deposits then thoroughly wet surfaces as required. Use 1 ounce per gallon of water for a minimum contact time of 10 minutes in a single application. Can be applied with a mop, sponge or cloth as well as spraying or soaking. The recommended use solution is prepared fresh for each use and then discarded. Rinsing is not necessary unless floors are to be waxed or polished. *Kills HIV-1 (AIDS Virus) on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surface/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission for human immunodeficiency virus Type-1 (HIV-1) (associated with AIDS). SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (AIDS VIRUS) OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.

Personal Protection: Disposable latex or vinyl gloves, gowns, face masks or eye coverings as appropriate, must be worn during all cleaning of body fluids, blood and decontamination procedures.

Cleaning Procedures: Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of disinfectant.

Contact Time: Effective against HIV-1 (AIDS Virus) on hard non-porous surfaces in the presence of a moderate amount of organic soil (5% blood serum) when used at a 1:128 dilution (providing 600 ppm of active quaternary) in 400ppm CaCO3 hard water for a contact time of 4 minutes at room temperature 20-25 degrees C.). Use a ten (10) minute contact time for disinfection against all other bacteria, fungi, and viruses claimed (listed on labeling).

Disposal of Infectious Material: Blood and other body fluids should be autoclaved and disposed of according to federal, state, and local authorities for infectious waste disposal.

Application:

- Immersion and Flow Cleaning
- Manual Cleaning



PRODUCT PROPERTIES

Color: None

Odor: Slight Chemical

Base: Quaternary Ammonium Chloride

pH (concentrate): 7.5-8.0

Viscosity: Water Thin

Foam: Moderate

Contact CANI, Inc. for technical services or product recommendations.

610.222.4500 • CANIUSA.COM

